




EHSP GUIDE TO GOWN SELECTION & USE

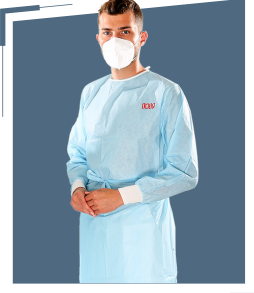

◀ AAMI BARRIER LEVELS ▶

LEVEL 1	LEVEL 2	LEVEL 3	LEVEL 4
---------	---------	---------	---------

BARRIER-LEVEL PROTECTION OF MEDICAL GOWNS

Innovation.Speed.Accuracy
for Your Protection

AAMI GOWN BARRIER LEVELS OF EHSP

The Association for the Advancement of Medical Instrumentation (AAMI) standards are designed to help medical professionals for the safe use of medical devices. AAMI introduced the voluntary standard ANSI/AAMI PB70:2012, Liquid Barrier Performance and Classification of Protective Apparel and Drapes Intended for Use in Health Care Facilities, to determine key identification measures for the appropriate selection of protective apparel and drapes, such as medical gowns, for use in healthcare facilities. The AAMI gown classification stems from four levels of barrier performance.

AAMI LEVEL 1 **Minimal Fluid Barrier Protection:** Used for daily patient care, when there is little to no risk of fluid exposure. Generally not used in the OR.

AAMI LEVEL 3 **Moderate Fluid Barrier Protection:** Used for the widest range of surgical procedures, where the risk of fluid exposure is moderate.

AAMI LEVEL 2 **Minimal to Low Fluid Barrier Protection:** Used when there's only a slight risk of fluid exposure, for minimally invasive surgical procedures such as lumps and bumps.

AAMI LEVEL 4 **Highest Fluid and Microbial Barrier Protection:** Provides protection against blood-borne pathogens in critical zones; used for long, fluid-intensive procedures.

STANDARDS FOR ANSI/AAMI PB70:2012 GOWNS

	LEVEL 1	LEVEL 2	LEVEL 3	LEVEL 4
BARRIER PROTECTION	Minimal	Low	Moderate	High
RESISTANCE MEASURES	Liquid Penetration	Liquid Penetration	Liquid Penetration	Liquid and Viral Penetration
TEST	AATCC42	AATCC42 AATCC127	AATCC42 AATCC127	ASTM F1671
TEST CRITERIA	Water Impact ≤ 4.5g	Spray Impact ≤ 1.0g Hydrostatic Pressure ≥ 20cm	Spray Impact ≤ 1.0g Hydrostatic Pressure ≥ 50cm	Pass
ACCEPTABLE QUALITY LEVEL	4%	Spray Impact 4% Hydrostatic Pr. 4%	Spray Impact 4% Hydrostatic Pr. 4%	4%

The requirements of the European Union's EN13795 standard for the barrier capacity of surgical gowns mainly include resistance to dry and wet microbes and resistance to liquid penetration. Surgical gowns are divided into standard surgical gowns and reinforced surgical gowns according to different purposes. Ordinary surgical gowns are used for routine surgery with less bleeding, and reinforced surgical clothes are used for surgery with longer operation time and more bleeding, such as orthopedic surgery and gastrointestinal surgery.

STANDARDS FOR EN 13795 SURGICAL CLOTHES

QUALITY INDEX	STANDARD SURGICAL GOWN		REINFORCED SURGICAL GOWN	
	KEY AREA	SUB-KEY AREA	KEY AREA	SUB-KEY AREA
RESISTANCE TO MICROBIAL PENETRATION (DRY STATE) (LOG10CFU)	Not Required	≤2	Not Required	≤2
RESISTANCE TO MICROBIAL PENETRATION (WET) (BI)	≥2.8	Not Required	6.0	Not Required
RESISTANCE TO LIQUID PENETRATION (CMH2O)	≥20	≥10	≥100	≥10

KNOW YOUR GOWNS

DEFINITION OF KEY TERMS:

Gown Selection Made Easy



Resistance to microbial penetration (dry state) (log10CFU) represents the bacterial liquid with a concentration of 10⁸ CFU/g, and shake the bacteria for 30 min. For the purpose of this standard, log10CFU ≤ 2 means a maximum of 300 CFU.




Resistance to microbial penetration (wet) (BI) represents the lowest significant difference (LSD) of BI reached 0.98 in the 95% confidence interval, which is the lowest value between the two materials, and the difference is not within 0.98BI. BI=6.0, Indicates no penetration in this standard.



Key area Strictly maintain the sterile area during operation, including the front part of the waist, the upper part, the arms.



Sub-key area Areas with less sterility during operation, including the back, front and lower waist.

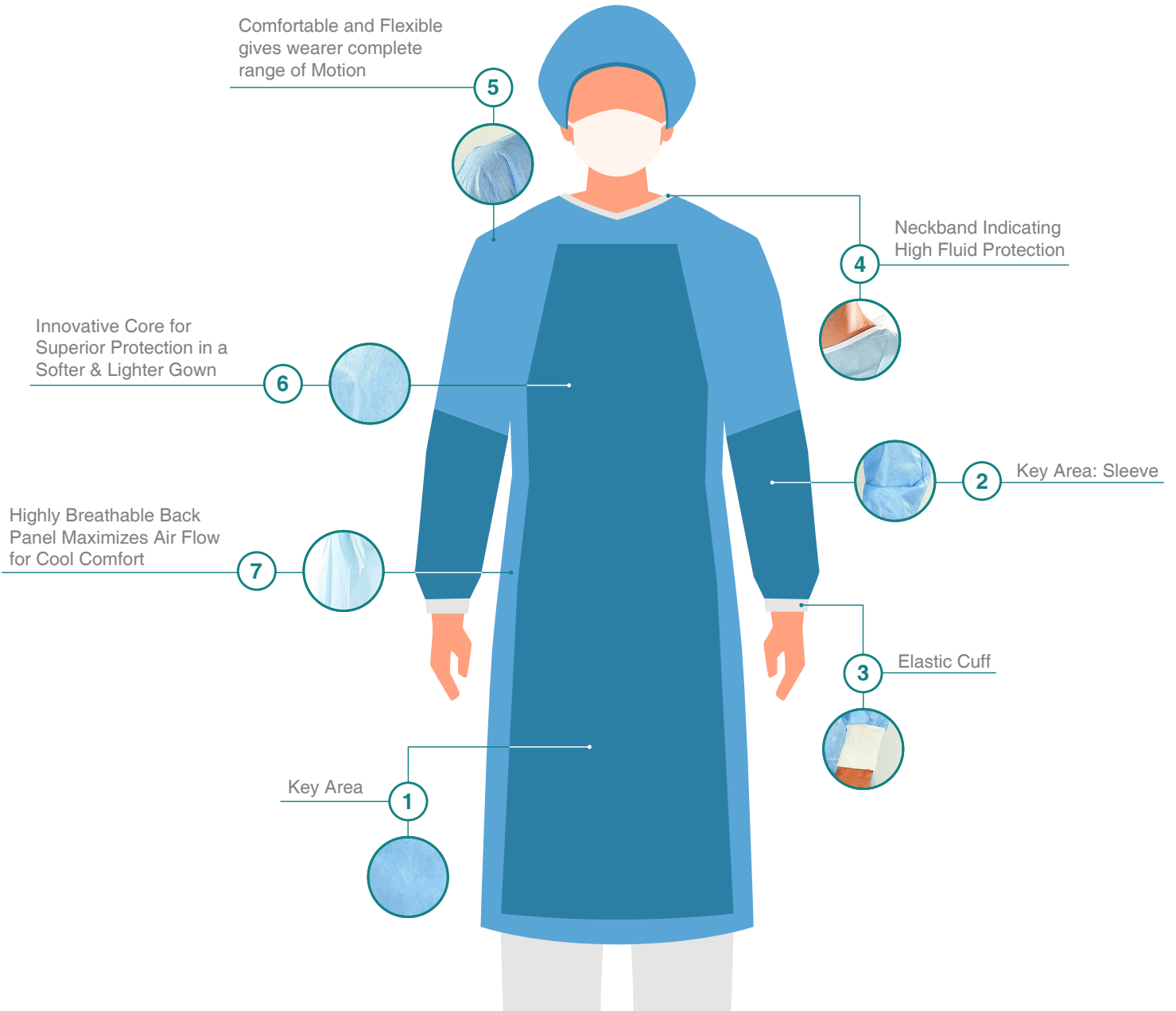



EHSP 800 Level-1 Isolation Gown

- PP+PE (35 GSM) Exceeding Level - I
- AAMI Level-1 Tested
- Adjustable Velcro Neck and Waist Ties
- 1" Extended Overlapping Taps for Easy Gripping
- Overlock Sewn
- Neck Part Comes with an Overlapping Cloth, & the Back Neck Tie Comes with 2" & 4" Adjustable Velcro



EHSP 810 Level-2 Isolation Gown

- PP+PE (40 GSM) Exceeding Level-2 requirements
- AAMI Level-2 Tested
- Adjustable Velcro Neck and Waist Ties
- 1" Extended Overlapping Taps for Easy Gripping
- Overlock Sewn - EN 14126 Standard Fabric
- Neck Part Comes with an Overlapping Cloth, & the Back Neck Tie Comes with 2" & 4" Adjustable Velcro



EHSP 820 Level-3 Isolation Gown

- PP+PE (50-60 GSM)
- AAMI Level-3 Tested
- Adjustable Velcro Neck and Waist Ties
- 1" Extended Overlapping Taps for Easy Gripping
- Overlock Sewn
- Neck Part Comes with an Overlapping Cloth, & the Back Neck Tie Comes with 2" & 4" Adjustable Velcro



EHSP 830 Level-3 Surgical Isolation Gown

- 40 SMS + 25 GSM Barrier (65 GSM)
- AAMI Level-3 Tested
- Adjustable Velcro Neck and Waist Ties
- 1" Extended Overlapping Taps for Easy Gripping
- Overlock Sewn
- Neck Part Comes with an Overlapping Cloth, & the Back Neck Tie Comes with 2" & 4" Adjustable Velcro



EHSP 840 Level-2 Isolation Gown

- 40 GSM SMS + 15 GSM Lamination + 3 GSM PE (58 GSM)
- AAMI Level-2 Tested
- Full Barrier Apron Laminated & Water & Fluid Splatter Protected with Full Sleeves
- Adjustable Velcro Neck and Waist Ties
- 1" Extended Overlapping Taps for Easy Gripping
- Overlock Sewn
- Neck Part Comes with an Overlapping Cloth, & the
- Back Neck Tie Comes with 2" & 4" Adjustable Velcro



EHSP 850 Chemotherapy Gown

- 80 GSM SMS with Lamination
- ASTM 739-12 & ASTM F1671 Certified - All Chemicals Tested
- Full Barrier Apron Laminated & Water & Fluid Splatter Protected with Full Sleeves
- Adjustable Velcro Neck and Waist Ties
- 1" Extended Overlapping Taps for Easy Gripping
- Overlock Sewn
- Microbial Barrier Protection to Protect the Wearer Against Longtime Exposures



EHSP 860 Level-1 & 2 Isolation Gown

- MD (32) - CD(45-75) $\pm 10\%$
- AAMI Level-1 & 2 Tested
- Adjustable Velcro Neck and Waist Ties
- 1" Extended Overlapping Taps for Easy Gripping
- Overlock Sewn
- Neck Part Comes with an Overlapping Cloth, & the Back Neck Tie Comes with 2" & 4" Adjustable Velcro




EHSP 900 Level-4 Reinforced Surgical Gown

- PP+PE (55-60 GSM) + 25 GSM BARRIER + Tape Reinforcement
- AAMI Level-4 Tested
- Full Barrier Apron Laminated & Water & Fluid Splatter Protected with Full Sleeves
- 3" Knitted Cuff, Elasticated Back Neck Velcro and Waist Tie-Back
- Overlock Sewn - Microbial Barrier Protection to Protect the Wearer Against Longtime Exposures

GOWN SIZE GUIDE

GOWN PARTS	Small	Medium	Large	X-Large	2X-Large	3X-Large	
Gown Length	110/43	112/44	114/45	127/50	129/51	131/52	Level 1
Chests Width	120/47	130/51	135/53	140/55	145/57	150/59	
Sleeve Length	52.5/20.5	55/21.5	57/22.5	63/24.8	64/25	65/25.5	
Chests Width	110/43	112/44	114/45	135/53	137/54	139/55	Level 2 & 3
Gown Length	105/41	115/45	125/49	127/50	129/51	140/55	
Sleeve Length	52.5/20.5	55/21.5	57/22.5	63/24.8	64/25	65/25.5	
Chests Width	110/43	112/44	114/45	127/50	129/51	131/52	Level 4 & Chemo Gown
Gown Length	125/49	135/53	140/55	150/59	155/61	160/63	
Sleeve Length	52.5/20.5	55/21.5	57/22.5	63/24.8	64/25	65/25.5	

Unit: cm/inch

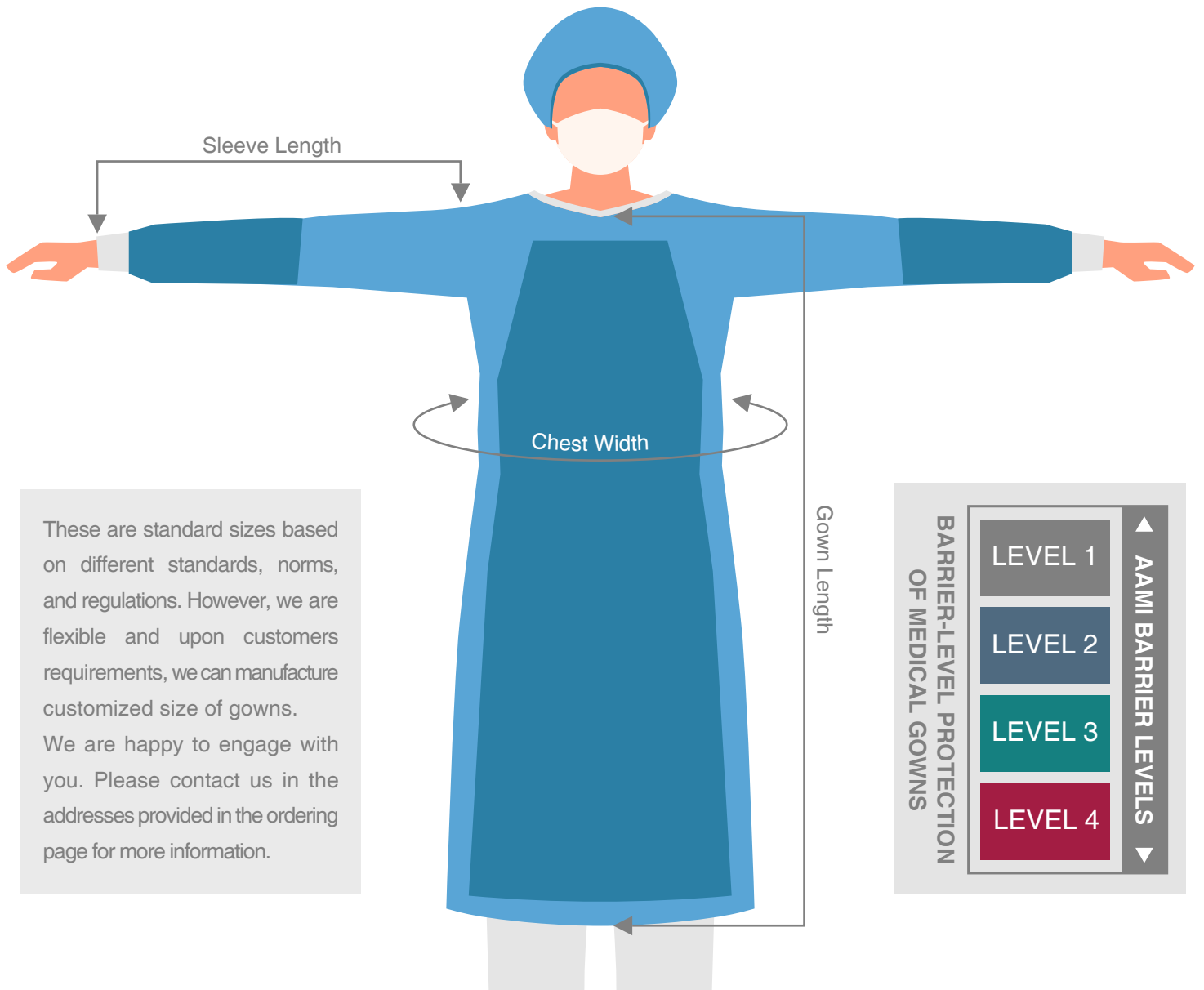

These are standard sizes based on different standards, norms, and regulations. However, we are flexible and upon customers requirements, we can manufacture customized size of gowns.

We are happy to engage with you. Please contact us in the addresses provided in the ordering page for more information.

BARRIER-LEVEL PROTECTION OF MEDICAL GOWNS

AAAMI BARRIER LEVELS

- LEVEL 1
- LEVEL 2
- LEVEL 3
- LEVEL 4

ORDERING INFORMATION

Description	Quantity	Non-Sterile	Sterile	Description	Quantity	Non-Sterile	Sterile
EHSP 800 Level-1 Isolation Gown				EHSP 830 Level-3 Surgical Isolation Gown			
S	ldv Pck	EHSP800-NS-S	EHSP800-ST-S	S	ldv Pck	EHSP830-NS-S	EHSP830-ST-S
M	ldv Pck	EHSP800-NS-M	EHSP800-ST-M	M	ldv Pck	EHSP830-NS-M	EHSP830-ST-M
L	ldv Pck	EHSP800-NS-L	EHSP800-ST-L	L	ldv Pck	EHSP830-NS-L	EHSP830-ST-L
XL	ldv Pck	EHSP800-NS-X	EHSP800-ST-X	XL	ldv Pck	EHSP830-NS-X	EHSP830-ST-X
2XL	ldv Pck	EHSP800-NS-2X	EHSP800-ST-2X	2XL	ldv Pck	EHSP830-NS-2X	EHSP830-ST-2X
3XL	ldv Pck	EHSP800-NS-3X	EHSP800-ST-3X	3XL	ldv Pck	EHSP830-NS-3X	EHSP830-ST-3X
EHSP 810 Level-2 Isolation Gown				EHSP 850 Chemotherapy Gown			
S	ldv Pck	EHSP810-NS-S	EHSP810-ST-S	S	ldv Pck	EHSP850-NS-S	EHSP850-ST-S
M	ldv Pck	EHSP810-NS-M	EHSP810-ST-M	M	ldv Pck	EHSP850-NS-M	EHSP850-ST-M
L	ldv Pck	EHSP810-NS-L	EHSP810-ST-L	L	ldv Pck	EHSP850-NS-L	EHSP850-ST-L
XL	ldv Pck	EHSP810-NS-X	EHSP810-ST-X	XL	ldv Pck	EHSP850-NS-X	EHSP850-ST-X
2XL	ldv Pck	EHSP810-NS-2X	EHSP810-ST-2X	2XL	ldv Pck	EHSP850-NS-2X	EHSP850-ST-2X
3XL	ldv Pck	EHSP810-NS-3X	EHSP810-ST-3X	3XL	ldv Pck	EHSP850-NS-3X	EHSP850-ST-3X
EHSP 840 Level-2 Isolation Gown				EHSP 900 Level-4 Reinforced Surgical Gown (White)			
S	ldv Pck	EHSP840-NS-S	EHSP840-ST-S	S	ldv Pck	EHSP900-NS-WE-S	EHSP900-ST-WE-S
M	ldv Pck	EHSP840-NS-M	EHSP840-ST-M	M	ldv Pck	EHSP900-NS-WE-M	EHSP900-ST-WE-M
L	ldv Pck	EHSP840-NS-L	EHSP840-ST-L	L	ldv Pck	EHSP900-NS-WE-L	EHSP900-ST-WE-L
XL	ldv Pck	EHSP840-NS-X	EHSP840-ST-X	XL	ldv Pck	EHSP900-NS-WE-X	EHSP900-ST-WE-X
2XL	ldv Pck	EHSP840-NS-2X	EHSP840-ST-2X	2XL	ldv Pck	EHSP900-NS-WE-2X	EHSP900-ST-WE-2X
3XL	ldv Pck	EHSP840-NS-3X	EHSP840-ST-3X	3XL	ldv Pck	EHSP900-NS-WE-3X	EHSP900-ST-WE-3X
EHSP 860 Level-1 & 2 Isolation Gown				EHSP 900 Level-4 Reinforced Surgical Gown (Blue)			
S	ldv Pck	EHSP860-NS-S	EHSP860-ST-S	S	ldv Pck	EHSP900-NS-BE-S	EHSP900-ST-BE-S
M	ldv Pck	EHSP860-NS-M	EHSP860-ST-M	M	ldv Pck	EHSP900-NS-BE-M	EHSP900-ST-BE-M
L	ldv Pck	EHSP860-NS-L	EHSP860-ST-L	L	ldv Pck	EHSP900-NS-BE-L	EHSP900-ST-BE-L
XL	ldv Pck	EHSP860-NS-X	EHSP860-ST-X	XL	ldv Pck	EHSP900-NS-BE-X	EHSP900-ST-BE-X
2XL	ldv Pck	EHSP860-NS-2X	EHSP860-ST-2X	2XL	ldv Pck	EHSP900-NS-BE-2X	EHSP900-ST-BE-2X
3XL	ldv Pck	EHSP860-NS-3X	EHSP860-ST-3X	3XL	ldv Pck	EHSP900-NS-BE-3X	EHSP900-ST-BE-3X
EHSP 820 Level-3 Isolation Gown (White)				EXPERIENCE OUR FLEXIBILITY BY SENDING YOUR CUSTOMIZED ORDERS.			
S	ldv Pck	EHSP820-NS-WE-S	EHSP820-ST-WE-S				
M	ldv Pck	EHSP820-NS-WE-M	EHSP820-ST-WE-M				
L	ldv Pck	EHSP820-NS-WE-L	EHSP820-ST-WE-L				
XL	ldv Pck	EHSP820-NS-WE-X	EHSP820-ST-WE-X				
2XL	ldv Pck	EHSP820-NS-WE-2X	EHSP820-ST-WE-2X				
3XL	ldv Pck	EHSP820-NS-WE-3X	EHSP820-ST-WE-3X	To Order Please Contact			
EHSP 820 Level-3 Isolation Gown (Blue)							
S	ldv Pck	EHSP820-NS-BE-S	EHSP820-ST-BE-S				
M	ldv Pck	EHSP820-NS-BE-M	EHSP820-ST-BE-M				
L	ldv Pck	EHSP820-NS-BE-L	EHSP820-ST-BE-L	contracting@ecephan.com			
XL	ldv Pck	EHSP820-NS-BE-X	EHSP820-ST-BE-X				
2XL	ldv Pck	EHSP820-NS-BE-2X	EHSP820-ST-BE-2X	+90 543 822 1717			
3XL	ldv Pck	EHSP820-NS-BE-3X	EHSP820-ST-BE-3X				
EHSP 820 Level-3 Isolation Gown (Blue)				www.ecephan.com			
S	ldv Pck	EHSP820-NS-BE-S	EHSP820-ST-BE-S				
M	ldv Pck	EHSP820-NS-BE-M	EHSP820-ST-BE-M				
L	ldv Pck	EHSP820-NS-BE-L	EHSP820-ST-BE-L				
XL	ldv Pck	EHSP820-NS-BE-X	EHSP820-ST-BE-X	Turkey Office: ECEP HAN MEDİKAL TEK. SAN. TİC LTD. ŞTİ AKÇABURGAZ MAHALLESİ AKÇABURGAZ CADDESİ NO 20/28 AKTİM 1 TİCARET VE İŞ MERKEZİ ESENYURT/ İSTANBUL ☎ +90 212 852 3447 ✉ turkey@ecephan.com			
2XL	ldv Pck	EHSP820-NS-BE-2X	EHSP820-ST-BE-2X				
3XL	ldv Pck	EHSP820-NS-BE-3X	EHSP820-ST-BE-3X	USA Office: ECEP HAN NORTH AMERICA LLC 8260 E. GELDING DR. SUITE 101, SCOTTSDALE AZ 85260, ARIZONA USA ☎ +1 480 656 5519 ✉ america@ecephan.com			
EHSP 820 Level-3 Isolation Gown (Blue)							
S	ldv Pck	EHSP820-NS-BE-S	EHSP820-ST-BE-S	Swiss Office: ECEP HAN MEDİKAL TEK. SAN. TİC LTD. ŞTİ 37 ROUTE DE VIRELOUP 1239 COLLEX SWITZERLAND ☎ +41 79 714 3173 ✉ swiss@ecephan.com			
M	ldv Pck	EHSP820-NS-BE-M	EHSP820-ST-BE-M				
L	ldv Pck	EHSP820-NS-BE-L	EHSP820-ST-BE-L	Dubai Office: ECEP HAN INTERNATIONAL GENERAL TRADING LLC TESLA BUSINESS CENTER OPAL TOWER, SUIT NO. 90, 16TH FLOOR 1610, BUSINESS BAY PO. BOX NO. 39354 ☎ +97 150 696 1828 \ +97 155 676 1828 ✉ dubai@ecephan.com			
XL	ldv Pck	EHSP820-NS-BE-X	EHSP820-ST-BE-X				
2XL	ldv Pck	EHSP820-NS-BE-2X	EHSP820-ST-BE-2X	Afghanistan: ECEP HAN TRADING LLC HOUSE NO 754, STREET NO. 14 DISTRICT 6, POSTAL CODE NO. 1006 KART-E-SAY ☎ +93 202 501 610 ✉ kabul@ecephan.com			
3XL	ldv Pck	EHSP820-NS-BE-3X	EHSP820-ST-BE-3X				

To Order Please Contact

✉ contracting@ecephan.com

☎ +90 543 822 1717

🌐 www.ecephan.com



▲ AAMI BARRIER LEVELS ▼

LEVEL 1

LEVEL 2

LEVEL 3

LEVEL 4



Turkey Office:
ECEP HAN MEDİKAL TEK. SAN. TİC LTD. ŞTİ
AKÇABURGAZ MAHALLESİ AKÇABURGAZ CADDESİ
NO 20/28 AKTİM 1 TİCARET VE İŞ MERKEZİ
ESENYURT/ İSTANBUL
☎ +90 212 852 3447
✉ turkey@ecephan.com



USA Office:
ECEP HAN NORTH AMERICA LLC
8260 E. GELDING DR. SUITE 101,
SCOTTSDALE AZ 85260, ARIZONA USA
☎ +1 480 656 5519
✉ america@ecephan.com



Swiss Office:
ECEP HAN MEDİKAL TEK. SAN. TİC LTD. ŞTİ
37 ROUTE DE VIRELOUP
1239 COLLEX SWITZERLAND
☎ +41 79 714 3173
✉ swiss@ecephan.com



Dubai Office:
ECEP HAN INTERNATIONAL GENERAL TRADING LLC
TESLA BUSINESS CENTER OPAL TOWER,
SUIT NO. 90, 16TH FLOOR 1610,
BUSINESS BAY PO. BOX NO. 39354
☎ +97 150 696 1828 \ +97 155 676 1828
✉ dubai@ecephan.com



Afghanistan:
ECEP HAN TRADING LLC
HOUSE NO 754, STREET NO. 14
DISTRICT 6, POSTAL CODE NO. 1006
KART-E-SAY
☎ +93 202 501 610
✉ kabul@ecephan.com

EHSP 800 LEVEL-1 ISOLATION GOWN

AAMI Level-1 Isolation Gown

Ecep Han
Trading for good..



PP+PE
(35 GSM)
Exceeding
Level - I



ISO 13485:2016
Certified



AAMI Level-I Tested



GENERAL DESCRIPTION

- Overlock Sewn
- Full Coverage with Overlapping
- The Neck Part Comes with an Overlapping Cloth, and the Back Neck Tie Comes with 2" and 4" Adjustable Velcro and has 1" Extended Overlapping Taps for Easy Gripping
- It is a Protective Procedure Cover Gown for Visitors or Standard Procedure Medical Units



AAMI Level-I Certified and Used for Basic Care with Minimal Risk



Disposable



Sterility
Non-Sterile



Gender
Unisex



Fastening Type
Adjustable Velcro Neck and Waist Ties



Private Labeling
Possible upon Request;
Subject to MOQ.



Size
S, M, L, XL, 2XL, 3XL



Packing Style
50 PCs & 100 PCs

RECOMMENDED TO:

- Low and Minimal Risk-Zones
- Standard Procedure Medical Units
- Visitors

SHIPMENT DETAILS AND SERVICES

Airway Shipment	Seaway Shipment	Roadway Shipment	Railway Shipment
EXW 2020	FOB 2020	CIF 2020	DAP 2020
DDP 2020	CFR 2020	(1000 x 1200 mm) (40" x 48") Europe, North America	(1100 x 1100 mm) (44" x 44") Asia
(800 x 1200 mm) (32" x 48") Europe			

EHSP[®]
Innovation.Speed.Accuracy
For Your Protection

Turkey Office:
ECEP HAN MEDİKAL TEK. SAN. TİC. LTD. ŞTİ
AKÇABURGAZ MAHALLESİ AKÇABURGAZ CADDESİ
NO 20/28 AKTİM 1 TİCARET VE İŞ MERKEZİ
ESENYURT/ İSTANBUL
+90 212 852 3447
turkey@ecephan.com

USA Office:
ECEP HAN NORTH AMERICA LLC
8260 E. GELDING DR. SUITE 101,
SCOTTSDALE AZ 85260, ARIZONA USA
+1 480 656 5519
america@ecephan.com

Swiss Office:
ECEP HAN MEDİKAL TEK. SAN. TİC. LTD. ŞTİ
37 ROUTE DE VIRELOUP
1239 COLLEX SWITZERLAND
+41 79 714 3173
swiss@ecephan.com

Dubai Office:
ECEP HAN INTERNATIONAL GENERAL TRADING LLC
TESLA BUSINESS CENTER OPAL TOWER,
SUITE NO. 90, 16th FLOOR, 1610,
BUSINESS BAY PO. BOX NO. 39354
+97 150 696 1828 \ +97 155 676 1828
dubai@ecephan.com

Afghanistan:
ECEP HAN TRADING LLC
HOUSE NO 754, STREET NO. 14
DISTRICT 6, POSTAL CODE NO. 1006
KARFE-SAY
+93 202 501 610
kابل@ecephan.com

EHSP 810 ISOLATION GOWN

AAMI Level-2 Isolation Gown

Ecep Han
Trading for good..



PP+PE
(40 GSM)
Exceeding
Level - 2
Requirements



EN 14126 Standard Fabric



Protected Against Infectious
Diseases & Micro-organisms



AAMI Level-2 Tested



GENERAL DESCRIPTION

- Over-lock Sewn
- Full Coverage with Overlapping
- The Neck Part Comes with an Overlapping Cloth, and the Back Neck Tie Comes with 2" and 4" Adjustable Velcro and Has 1" Extended Overlapping Taps for Easy Gripping
- Provides a Barrier Against Large Amount of Penetration through Splatter and Against Some Fluid Exposures through Soaking
- It is Used as a Protective Procedure Gown, Standard Isolation, and in Medical Units



AAMI Level-II Certified and Provides a Barrier Against Large Amount of Penetration



Disposable



Sterility
Non-Sterile



Gender
Unisex



Fastening Type
Adjustable Velcro Neck and Waist Ties



Private Labeling
Possible upon Request;
Subject to MOQ.



Size
S, M, L, XL, 2XL, 3XL



Packing Style
50 PCs & 100 PCs

RECOMMENDED TO:

- Low and Minimal Risk-Zones
- Standard Procedure Medical Units
- Visitors

SHIPMENT DETAILS AND SERVICES

		Standard
Airway Shipment	EXW 2020 FOB 2020 CIF 2020 DAP 2020 DDP 2020 CFR 2020	(1000 x 1200 mm) (40" x 48") Europe, North America
Seaway Shipment		(1100 x 1100 mm) (44" x 44") Asia
Roadway Shipment		(800 x 1200 mm) (32" x 48") Europe
Railway Shipment		

EHSP[®]
Innovation.Speed.Accuracy
For Your Protection

Turkey Office:
ECEP HAN MEDİKAL TEK. SAN. TİC. LTD. ŞTİ
AKÇABURGAZ MAHALLESİ AKÇABURGAZ CADDESİ
NO 20/28 AKTİM 1 TİCARET VE İŞ MERKEZİ
ESENYURT/ İSTANBUL
☎ +90 212 852 3447
✉ turkey@ecephan.com

USA Office:
ECEP HAN NORTH AMERICA LLC
8260 E. GELDING DR. SUITE 101,
SCOTTSDALE AZ 85260, ARIZONA USA
☎ +1 480 656 5519
✉ america@ecephan.com

Swiss Office:
ECEP HAN MEDİKAL TEK. SAN. TİC. LTD. ŞTİ
37 ROUTE DE VIRELOUP
1239 COLLEX SWITZERLAND
☎ +41 79 714 3173
✉ swiss@ecephan.com

Dubai Office:
ECEP HAN INTERNATIONAL GENERAL TRADING LLC
TESLA BUSINESS CENTER OPAL TOWER,
SUITE NO. 90, 16th FLOOR 1610,
BUSINESS BAY PO. BOX NO. 39354
☎ +97 150 696 1828 \ +97 155 676 1828
✉ dubai@ecephan.com

Afghanistan:
ECEP HAN TRADING LLC
HOUSE NO 754, STREET NO. 14
DISTRICT 6, POSTAL CODE NO. 100
KARFE-SAY
☎ +93 202 501 610
✉ kabul@ecephan.com

EHSP 820-3 SURGICAL GOWN

AAMI Level-3 Surgical Gown

Ecep Han
Trading for good..



PP+PE
(50-60 GSM)



ISO 13485:2016
Certified



AAMI Level-3 Tested



Provides a Barrier Against
Large Amount of Penetration
Through Splatter



GENERAL DESCRIPTION

- Over-lock Sewn
- Full Coverage with Overlapping
- The Neck Part Comes with an Overlapping Cloth, and the Back Neck Tie Comes with 2" and 4" Adjustable Velcro and has 1" Extended Overlapping Taps for Easy Gripping
- Used in Moderate Risk-Zones for Aerial Blood Draw and Inserting an Intravenous (IV) Line in Emergency Rooms, Medical Units, and Trauma Cases



AAMI Level-III Certified and
Used in Moderate Risk-Zones



Disposable



Sterility
Non-Sterile; Sterilized
upon Request



Gender
Unisex



Fastening Type
Adjustable Velcro Neck and
Waist Ties



Private Labeling
Possible upon Request;
Subject to MOQ.



Size
S, M, L, XL, 2XL, 3XL



Packing Style
40 PCs & 80 PCs

RECOMMENDED TO:

- Emergency Rooms
- ICUs
- Trauma Cases

SHIPMENT DETAILS AND SERVICES

Airway Shipment	Seaway Shipment	EXW 2020 FOB 2020 CIF 2020 DAP 2020 DDP 2020 CFR 2020	(1000 x 1200 mm) (40" x 48") Europe, North America (1100 x 1100 mm) (44" x 44") Asia (800 x 1200 mm) (32" x 48") Europe

EHSP[®]
Innovation.Speed.Accuracy
For Your Protection

Turkey Office:
ECEP HAN MEDİKAL TEK. SAN. TİC. LTD. ŞTİ
AKÇABURGAZ MAHALLESİ AKÇABURGAZ CADDESİ
NO 20/28 AKTİM 1 TİCARET VE İŞ MERKEZİ
ESENYURT/ İSTANBUL
☎ +90 212 852 3447
✉ turkey@ecephan.com

USA Office:
ECEP HAN NORTH AMERICA LLC
8260 E. GELDING DR. SUITE 101,
SCOTTSDALE AZ 85260, ARIZONA USA
☎ +1 480 656 5519
✉ america@ecephan.com

Swiss Office:
ECEP HAN MEDİKAL TEK. SAN. TİC. LTD. ŞTİ
37 ROUTE DE VIRELOUP
1239 COLLEX SWITZERLAND
☎ +41 79 714 3173
✉ swiss@ecephan.com

Dubai Office:
ECEP HAN INTERNATIONAL GENERAL TRADING LLC
TESLA BUSINESS CENTER OPAL TOWER,
SUITE NO. 90, 16th FLOOR, 1610,
BUSINESS BAY PO. BOX NO. 39354
☎ +97 150 696 1828 \ +97 155 676 1828
✉ dubai@ecephan.com

Afghanistan:
ECEP HAN TRADING LLC
HOUSE NO 754, STREET NO. 14
DISTRICT 6, POSTAL CODE NO. 1006
KARFE-SAY
☎ +93 202 501 610
✉ kabul@ecephan.com

EHSP 830-1 SURGICAL GOWN

AAMI Level-3 Surgical Gown

Ecep Han
Trading for good..



40 SMS
+
25 GSM
Barrier
(65 GSM)



Provides Barrier Against Some Fluid Exposures Through Soaking



AAMI Level-3 Tested



Provides a Barrier Against Large Amount of Penetration Through Splatter



GENERAL DESCRIPTION

- Full barrier Apron and Full Sleeves
- Over-lock Sewn
- Full Coverage with Overlapping
- The Neck Part Comes with an Overlapping Cloth, and the Back Neck Tie Comes with 2" and 4" Adjustable Velcro and has 1" Extended Overlapping Taps for Easy Gripping
- Used in Moderate Risk-Zones for Aerial Blood Draw and Inserting an Intravenous (IV) Line in Emergency Rooms, Medical Units, and Trauma Cases



AAMI Level-III Certified and Used in Moderate Risk Zones



Disposable



Sterility
Non-Sterile; Sterilized upon Request



Gender Unisex



Fastening Type
Adjustable Velcro Neck and Waist Ties



Private Labeling
Possible upon Request; Subject to MOQ.



Size
S, M, L, XL, 2XL, 3XL



Packing Style
32 PCs & 65 PCs

RECOMMENDED TO:

- Medium to High Risk-Zones
- Healthcare Personnel During Surgical Procedures
- ICUs

SHIPMENT DETAILS AND SERVICES

Airway Shipment	Seaway Shipment	EXW 2020 FOB 2020 CIF 2020 DAP 2020 DDP 2020 CFR 2020	(1000 x 1200 mm) (40" x 48") Europe, North America (1100 x 1100 mm) (44" x 44") Asia (800 x 1200 mm) (32" x 48") Europe

EHSP[®]
Innovation.Speed.Accuracy
For Your Protection

Turkey Office:
ECEP HAN MEDİKAL TEK. SAN. TİC. LTD. ŞTİ
AKÇABURGAZ MAHALLESİ AKÇABURGAZ CADDESİ
NO 20/28 AKTİM 1 TİCARET VE İŞ MERKEZİ
ESENYURT/ İSTANBUL
☎ +90 212 852 3447
✉ turkey@ecephan.com

USA Office:
ECEP HAN NORTH AMERICA LLC
8260 E. GELDING DR. SUITE 101,
SCOTTSDALE AZ 85260, ARIZONA USA
☎ +1 480 656 5519
✉ america@ecephan.com

Swiss Office:
ECEP HAN MEDİKAL TEK. SAN. TİC. LTD. ŞTİ
37 ROUTE DE VIRELOUP
1239 COLLEX SWITZERLAND
☎ +41 79 714 3173
✉ swiss@ecephan.com

Dubai Office:
ECEP HAN INTERNATIONAL GENERAL TRADING LLC
TESLA BUSINESS CENTER OPAL TOWER,
SUITE NO. 90, 16TH FLOOR, 1610,
BUSINESS BAY PO. BOX NO. 39354
☎ +97 150 696 1828 \ +97 155 676 1828
✉ dubai@ecephan.com

Afghanistan:
ECEP HAN TRADING LLC
HOUSE NO 754, STREET NO. 14
DISTRICT 6, POSTAL CODE NO. 1006
KARFE-SAY
☎ +93 202 501 610
✉ kabul@ecephan.com

EHSP 840-3 ISOLATION GOWN

AAMI Level-2 Isolation Gown

Ecep Han
Trading for good..



40 GSM
SMS
+
15 GSM
Lamination
+
3 GSM
PE (58 GSM)



Provides a Barrier Against Large Amount of Penetration Through Splatter



Provides Barrier Against Some Fluid Exposures Through Soaking



AAMI Level-2 Tested



GENERAL DESCRIPTION

- Full Barrier Apron Laminated
- Water and Fluid Splatter Protected with Full Sleeves
- Over-lock Sewn
- Full Coverage with Overlapping
- Neck Part Comes with Overlapping Cloth, and Back Neck Tie Comes with 2" and 4" Adjustable Velcro and has 1" Extended Overlapping Taps for Easy Gripping
- Provides a Barrier Against Large Amount of Penetration through Splatter and Against Some Fluid Exposures through Soaking. It is Used as a Protective Procedure Gown, Standard Isolation, and in Medical Units.



AAMI Level-II Certified and Provides a Barrier Against Large Amount of Penetration



Disposable



Sterility
Non-Sterile; Sterilized upon Request



Gender Unisex



Fastening Type
Adjustable Velcro Neck and Waist Ties



Private Labeling
Possible upon Request; Subject to MOQ.



Size
Standard Free Size



Packing Style
50 PCs & 100 PCs

RECOMMENDED TO:

- Medium to High Risk-Zones
- Healthcare Personnel

SHIPMENT DETAILS AND SERVICES

+		
Airway Shipment	EXW 2020 FOB 2020 CIF 2020 DAP 2020 DDP 2020 CFR 2020	Standard (1000 x 1200 mm) (40" x 48") Europe, North America (1100 x 1100 mm) (44" x 44") Asia (800 x 1200 mm) (32" x 48") Europe

EHSP[®]
Innovation.Speed.Accuracy
For Your Protection

Turkey Office:
ECEP HAN MEDİKAL TEK. SAN. TİC. LTD. ŞTİ
AKÇABURGAZ MAHALLESİ AKÇABURGAZ CADDESİ
NO 20/28 AKTİM 1 TİCARET VE İŞ MERKEZİ
ESENYURT/ İSTANBUL
☎ +90 212 852 3447
✉ turkey@ecephan.com

USA Office:
ECEP HAN NORTH AMERICA LLC
8260 E. GELDING DR. SUITE 101,
SCOTTSDALE AZ 85260, ARIZONA USA
☎ +1 480 656 5519
✉ america@ecephan.com

Swiss Office:
ECEP HAN MEDİKAL TEK. SAN. TİC. LTD. ŞTİ
37 ROUTE DE VIRELOUP
1239 COLLEX SWITZERLAND
☎ +41 79 714 3173
✉ swiss@ecephan.com

Dubai Office:
ECEP HAN INTERNATIONAL GENERAL TRADING LLC
TESLA BUSINESS CENTER OPAL TOWER,
SUITE NO. 90, 16TH FLOOR, 1610,
BUSINESS BAY PO. BOX NO. 39354
☎ +97 150 696 1828 \ +97 155 676 1828
✉ dubai@ecephan.com

Afghanistan:
ECEP HAN TRADING LLC
HOUSE NO 754, STREET NO. 14
DISTRICT 6, POSTAL CODE NO. 1006
KARFE-SAY
☎ +93 202 501 610
✉ kabul@ecephan.com

EHSP 850 CHEMOTHERAPY GOWN

Disposable Chemotherapy Gown (ASTM 739-12), All Chemicals Tested

Ecep Han
Trading for good..



80 GSM
SMS
WITH
LAMINATION



High Protection Against All Tested Chemicals



Barrier Against Large Amounts of Fluid Exposure over Long Periods



Barrier Against Blood Containing Virus, Pathogen & Infectious Diseases (Non-Airborne)



GENERAL DESCRIPTION

- Full Barrier Apron Laminated
- Water and Fluid Splatter Protected with Full sleeves
- Over-lock Sewn
- Full Coverage with Overlapping
- Neck Part Comes with Overlapping Cloth, and Back Neck Tie Comes with 2," and 4" Adjustable Velcro and has 1" Extended Overlapping Taps for Easy Gripping
- Microbial Barrier Protection to Protect the User Against Longtime Exposures
- Undergone ASTM F739-12 Standard Test Method for Permeation of Liquids and Gases Through Protective Clothing Materials Under Conditions of Continuous Contact
- Barrier Level Performance is Tested with Simulated Blood Containing a Virus, Pathogen Resistance, Infectious Diseases



ASTM F739-12/ ASTM F1671 Certified and Used in High-Risk Situations



Disposable



Sterility
Non-Sterile; Sterilized upon Request



Gender
Unisex



Fastening Type
Adjustable Velcro Neck and Waist Ties



Private Labeling
Possible upon Request; Subject to MOQ.



Size
S, M, L, XL, 2XL, 3XL



Packing Style
40 PCs & 80 PCs

RECOMMENDED TO:

- High Risk-Zones
- Healthcare Personnel
- Chemotherapy Purpose

SHIPMENT DETAILS AND SERVICES

Airway Shipment	Seaway Shipment	EXW 2020 FOB 2020 CIF 2020 DAP 2020 DDP 2020 CFR 2020	(1000 x 1200 mm) (40" x 48") Europe, North America (1100 x 1100 mm) (44" x 44") Asia (800 x 1200 mm) (32" x 48") Europe
Roadway Shipment	Railway Shipment		

EHSP[®]
Innovation.Speed.Accuracy
For Your Protection

Turkey Office:
ECEP HAN MEDİKAL TEK. SAN. TİC. LTD. ŞTİ
AKÇABURGAZ MAHALLESİ AKÇABURGAZ CADDESİ
NO 20/28 AKTİM 1 TİCARET VE İŞ MERKEZİ
ESENYURT/ İSTANBUL
☎ +90 212 852 3447
✉ turkey@ecephan.com

USA Office:
ECEP HAN NORTH AMERICA LLC
8260 E. GELDING DR. SUITE 101,
SCOTTSDALE AZ 85260, ARIZONA USA
☎ +1 480 656 5519
✉ america@ecephan.com

Swiss Office:
ECEP HAN MEDİKAL TEK. SAN. TİC. LTD. ŞTİ
37 ROUTE DE VIRELOUP
1239 COLLEX SWITZERLAND
☎ +41 79 714 3173
✉ swiss@ecephan.com

Dubai Office:
ECEP HAN INTERNATIONAL GENERAL TRADING LLC
TESLA BUSINESS CENTER OPAL TOWER,
SUITE NO. 90, 16th FLOOR, 1610,
BUSINESS BAY PO. BOX NO. 39354
☎ +97 150 696 1828 \ +97 155 676 1828
✉ dubai@ecephan.com

Afghanistan:
ECEP HAN TRADING LLC
HOUSE NO 754, STREET NO. 14
DISTRICT 6, POSTAL CODE NO. 1006
KARFE-SAY
☎ +93 202 501 610
✉ kabul@ecephan.com

EHSP 860 L-1 & L-2 ISOLATION GOWN

AAMI Level-1 and Level-2 Isolation Gown

Ecep Han
Trading for good..



MD (32) - CD(45-75)
±10%



Provides a Barrier Against Large Amount of Penetration Through Splatter



AAMI Level-2 Tested



AAMI Level-1 Tested



GENERAL DESCRIPTION

- Over-lock Sewn
- Full Coverage with Overlapping
- The Neck Part Comes with an Overlapping Cloth, and the Back Neck Tie Comes with 2" and 4" Adjustable Velcro and has 1" Extended Overlapping Taps for Easy Gripping
- Protective Procedure Cover Gown for Visitors or Standard Procedure Medical Units



AAMI Level-I & Level-II Certified & Used for Basic Care with Minimal Risk.



Disposable



Sterility
Non-Sterile



Gender
Unisex



Fastening Type
Adjustable Velcro Neck and Waist Ties



Private Labeling
Possible upon Request; Subject to MOQ.



Size
S, M, L, XL, 2XL, 3XL



Packing Style
50 PCs & 100 PCs

RECOMMENDED TO:

- Basic Care (Hospital Workers, Care Homes, or Low-Risk Wards)
- Hospital Visitors
- Hotels Cleanliness

SHIPMENT DETAILS AND SERVICES

Airway Shipment	Seaway Shipment	EXW 2020 FOB 2020 CIF 2020 DAP 2020 DDP 2020 CFR 2020	(1000 x 1200 mm) (40" x 48") Europe, North America (1100 x 1100 mm) (44" x 44") Asia (800 x 1200 mm) (32" x 48") Europe

EHSP[®]
Innovation.Speed.Accuracy
For Your Protection

Turkey Office:
ECEP HAN MEDİKAL TEK. SAN. TİC. LTD. ŞTİ
AKÇABURGAZ MAHALLESİ AKÇABURGAZ CADDESİ
NO 20/28 AKTİM 1 TİCARET VE İŞ MERKEZİ
ESENYURT/ İSTANBUL
+90 212 852 3447
turkey@ecephan.com

USA Office:
ECEP HAN NORTH AMERICA LLC
8260 E. GELDING DR. SUITE 101,
SCOTTSDALE AZ 85260, ARIZONA USA
+1 480 656 5519
america@ecephan.com

Swiss Office:
ECEP HAN MEDİKAL TEK. SAN. TİC. LTD. ŞTİ
37 ROUTE DE VIRELOUP
1239 COLLEX SWITZERLAND
+41 79 714 3173
swiss@ecephan.com

Dubai Office:
ECEP HAN INTERNATIONAL GENERAL TRADING LLC
TESLA BUSINESS CENTER OPAL TOWER,
SUITE NO. 90, 16TH FLOOR, 1610,
BUSINESS BAY PO. BOX NO. 39354
+97 150 696 1828 \ +97 155 676 1828
dubai@ecephan.com

Afghanistan:
ECEP HAN TRADING LLC
HOUSE NO 754, STREET NO. 14
DISTRICT 6, POSTAL CODE NO. 1006
KARFE-SAY
+93 202 501 610
kابل@ecephan.com

EHSP 900 L-4 REINFORCED SURGICAL GOWN

AAMI Level-4 Reinforced Surgical Gown

Ecep Han
Trading for good..



PP+PE
(55-60 GSM)
+
25 GSM
BARRIER
+
Tape
Reinforcement



Provides Barrier Against Large Amount of Penetration Through Splatter



Barrier Against Blood Containing Virus, Pathogen & Infectious Diseases Using Phi-X174 Bacteriophage Penetration



AAMI Level-4 Tested



GENERAL DESCRIPTION

- Full Barrier Apron Laminated
- Water and Fluid Splatter Protected with Full sleeves
- Over-lock Sewn
- Full Coverage with Overlapping
- Neck Part Comes with Overlapping Cloth, Back Neck Tie Comes with 2," and 4" Adjustable Velcro and has 1" Extended Overlapping Taps for Easy Gripping
- Microbial Barrier Protection to Protect the User Against Longtime Contacting Blood Borne Pathogens
- ASTM F1671 Standard Test Method for the Resistance of Materials Used in Protective Clothing to Penetrate Blood-Borne Pathogens Using Phi-X174 Bacteriophage Penetration as a Test System



ANSI/AAMI Level -IV, ASTM F1671 Certified and Used in High-Risk Situations



Disposable



Sterility
Non-Sterile; Sterilized upon Request



Gender Unisex



Fastening Type
3" Knitted Cuff, Elasticated Back Neck Velcro and Waist Tie-Back



Private Labeling
Possible upon Request; Subject to MOQ.



Size
S, M, L, XL, 2XL, 3XL



Packing Style
32 PCs & 65 PCs

RECOMMENDED TO:

- High Risk-Zones
- Healthcare Personnel
- Surgical Purpose

SHIPMENT DETAILS AND SERVICES

Airway Shipment	Seaway Shipment	EXW 2020 FOB 2020 CIF 2020 DAP 2020 DDP 2020 CFR 2020	(1000 x 1200 mm) (40" x 48") Europe, North America (1100 x 1100 mm) (44" x 44") Asia (800 x 1200 mm) (32" x 48") Europe
Roadway Shipment	Railway Shipment		

EHSP[®]
Innovation.Speed.Accuracy
For Your Protection

Turkey Office:
ECEP HAN MEDİKAL TEK. SAN. TİC. LTD. ŞTİ
AKÇABURGAZ MAHALLESİ AKÇABURGAZ CADDESİ
NO 20/28 AKTİM 1 TİCARET VE İŞ MERKEZİ
ESENYURT/ İSTANBUL
☎ +90 212 852 3447
✉ turkey@ecephan.com

USA Office:
ECEP HAN NORTH AMERICA LLC
8260 E. GELDING DR. SUITE 101,
SCOTTSDALE AZ 85260, ARIZONA USA
☎ +1 480 656 5519
✉ america@ecephan.com

Swiss Office:
ECEP HAN MEDİKAL TEK. SAN. TİC. LTD. ŞTİ
37 ROUTE DE VIRELOUP
1239 COLLEX SWITZERLAND
☎ +41 79 714 3173
✉ swiss@ecephan.com

Dubai Office:
ECEP HAN INTERNATIONAL GENERAL TRADING LLC
TESLA BUSINESS CENTER OPAL TOWER,
SUITE NO. 90, 16TH FLOOR 1610,
BUSINESS BAY PO. BOX NO. 39354
☎ +97 150 696 1828 \ +97 155 676 1828
✉ dubai@ecephan.com

Afghanistan:
ECEP HAN TRADING LLC
HOUSE NO 754, STREET NO. 14
DISTRICT 6, POSTAL CODE NO. 1006
KARFE-SAY
☎ +93 202 501 610
✉ kabul@ecephan.com